



DentaLife

FerriStat

Hemostatic Gel

Composition	20% w/w Ferric Sulfate
Uses	FerriStat hemostatic gel is a powerful hemostatic agent which effectively eliminates various degrees of oral capillary bleeding and sulcular fluids during dental treatments. Recommended for posterior restorations and for use during crown preparation where bleeding needs to be controlled.
Advantages	<ul style="list-style-type: none"> • Provides significant hemostasis response • Rapid action • Eliminates sulcular fluid contamination • Convenient syringe delivery and refill system
Contraindications	Product is contraindicated for patients allergic to Ferric Sulfate. Thoroughly remove any residue of FerriStat gel from the preparation site to avoid compromising the setting of any impression materials or bonds.
Precautions & Warnings	Do not combine with other hemostatic agents or other materials without first thoroughly removing residues from the tooth and surrounding tissue. FerriStat is designed for intraoral use only. Before applying intraorally, verify flow from the syringe and tip. If there is resistance, replace tip and re-verify flow. Use recommended infusor tips. To avoid cross-contamination, do not reuse tips. If using a prefilled syringe, a syringe cover should be used if multiple uses of the syringe are intended. The syringe cover and tip should be disposed of after use.
Clinical Procedure	<ol style="list-style-type: none"> 1. Rub FerriStat hemostatic gel against the bleeding tissues until bleeding stops. 2. If using retraction cord, pack cord into the sulcus area around the tooth and leave for 1-3 minutes. 3. Remove retraction cord and spray with air/water. If bleeding persists, rub FerriStat gel onto the sulcus area again and leave for 1-3 minute. 4. Rinse off FerriStat gel, dry and prepare for impression
Poison Schedule	Not Applicable
Safety Directions	Do not use on patients with known allergies or sensitivity to Ferric Sulfate
First Aid	For advice, contact a Poisons Information Centre (13 11 26 for Australia; 0800 764 766 for New Zealand) or a doctor (at once)
Regulatory	ARTG 403251
Presentations	DL2580 10 x 1.2mL Syringes and 20 infusor tips DL2585 30mL Syringe
Storage	Store Below 30 °C
Shelf Life	3 years from date of manufacture as indicated on pack.



Developed and Manufactured in Australia by: Dentalife Australia Pty Ltd

9/505 Maroondah Hwy Ringwood, Vic 3134

www.dentalife.com.au

Version 06/2023 V1.1

