



DentaLife Clinicare

Medical Enzyme Cleaner

Composition	Enzymes, Surfactants, Anti-foaming agents.
Uses	Clinicare Medical Enzyme Cleaner is an enzymatic cleaner used for removing blood and other protein deposits from medical and dental instruments, prior to reprocessing and sterilisation.
Advantages	<ul style="list-style-type: none">• Effective cleaning with multiple enzymes• Neutral pH• Low foaming• Fragrance free
Precautions	<i>Wear recommended PPE.</i>
Clinical Procedure	<p>Manual Cleaning: 1. Rinse instruments with warm water to remove visible soil and organic material. 2. Dilute 5 mL per litre of warm water (30-35 °C). 3. Fully immerse instruments in the prepared solution for at least 1 minute. If heavy soiling is present, continue to soak instruments until visibly clean. Use a soft brush for light mechanical cleaning if necessary. 4. Rinse instruments thoroughly with warm water and visually inspect instruments to ensure cleanliness. 5. Prepare instruments for disinfection/sterilisation. 6. Dispose of diluted solution after use.</p> <p>Ultrasonic Cleaning: 1. Rinse instruments with warm water to remove visible soil and organic material. 2. Fill the ultrasonic tank with warm water (30-35 °C) and add Clinicare Medical Enzyme Cleaner in the dilution of 5 mL per litre. 3. Place the instruments in the ultrasonic chamber, ensuring they are fully submerged. 4. Run the ultrasonic cycle for the recommended cleaning time, lid must be on during operation. 5. Remove the instruments and rinse in warm water. 6. Prepare instruments for disinfection/sterilisation. 7. Solution should be made up daily and changed a minimum of twice daily or when solution is visibly cloudy.</p>
Poison Schedule	Not Applicable
Safety Directions	Avoid inhaling mist, vapours or spray. Avoid contact with skin. Avoid contact with eyes. Wash hands thoroughly after use. Store product in original packaging and ensure cap is tightly closed
First Aid	For advice, contact a Poisons Information Centre (Australia 13 11 26, New Zealand 0800 764 766) or call a doctor at once. If swallowed, do NOT induce vomiting. If in eyes, wash out immediately with water. Remove contact lenses, if present and easy to do so and continue rinsing. If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water.
Regulatory	ARTG 509613
Presentations	DL2201 – Clinicare Medical Enzyme Cleaner 5L
Storage	Store below 30 °C in a cool, dry place.
Shelf Life	2 years

