



Clinicare Suction Cleaner MR

Composition	Sulfamic Acid
Uses	Concentrated acid based liquid for simple preparation of monthly and intense treatment solutions for flushing of <ul style="list-style-type: none"> • Aspiration systems • Amalgam separators • Spittoons
Advantages	<ul style="list-style-type: none"> • Four actions – cleans, sanitises, decontaminates and deodorises • No disruptive foam formation • Economical concentrate presentation <ul style="list-style-type: none"> ~ Active at 4% dilution for intense treatment • Aldehyde free, Phenol free, Chlorine free • Safe and simple to use
Contraindications	Not to be mixed with any disinfectant or other cleaning agents.
Precautions	Wear protective gloves and garments and protect the eyes and face. Wash your hands after handling. Keep the product in tightly closed container in a cool, dry place.
Clinical Procedure	<p>Monthly / Intense Treatment</p> <ol style="list-style-type: none"> 1. Prepare 4% solution. Dispense 40 mL of Clinicare Suction Cleaner MR concentrate into a 1 L jug and make to volume with warm water, 50-60°C (preferred). 2. Aspirate the prepared solution whenever necessary but preferably at the end of the day. 3. Aspirate via the high and low volume suction tubes. 4. Leave the system for at least 1 hour after treatment and preferably overnight. 5. Rinse thoroughly with 2 litres of clean water prior to reuse. <p>Spittoon</p> <p>The spittoon should be sanitised and cleaned with a minimum of 250 mL prepared solution at the same time as the aspirator treatment.</p>
Poison Schedule	Schedule 5 according to SUSMP (Australia)
Warning	Corrosive
Safety Directions	Avoid contact with eyes. Avoid contact with skin.
First Aid	For advice, contact a Poisons Information Centre (Australia) 13 11 26, (Thailand) +66 (0)2 547 0100. If in eyes, hold eyelids apart and flush the eyes continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor or for at least 15 minutes. If skin or hair contact occurs, remove contaminated clothing and flush skin and hair with running water. If swallowed, do NOT induce vomiting.
Regulatory	Australia : ARTG No. 330306 Thailand : Manufacturing License No. Sor Phor 15/2558
Presentations	1 L Bottle Product Code : DL2741 5 L Bottle Product Code : DL2745
Storage	Store below 30°C in a cool, dry place.
Shelf Life	3 years from date of manufacture as indicated on pack.