UNIFAST™ III

SELF-CURING ACRYLIC RESIN FOR TEMPORARY INLAYS, CROWNS, BRIDGES AND REPAIRS

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Construction of temporary inlays, crowns and bridges.
- Repair of fractured dentures.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE

A. TEMPORARY INLAYS, CROWNS AND BRIDGES

- Direct Method – single shade standard technique
- 1) Take an impression prior to tooth preparation
- 2) Coat the prepared teeth and gingival tissue with a thin film of vaseline.
- 3) Dispense the desired amount of UNIFAST III powder in a rubber cup and add liquid, quickly mix with a plastic spatula for 10-15 seconds. Immediately pour the mixture into the intended area of the impression. Standard powder to liquid ratio is 1 g of powder to 0.5 mL of liquid (1 scale of powder to 1 scale of liquid). Powder to liquid ratio is adjustable between 2 g : 1 mL and 2 g : 1.5 mL.
- 4) Wait until the mixture reaches a soft dough stage.
- 5) Place the impression in the mouth with pressure.

NOTE:
UNIFAST III starts to set 1 minute after mixing. Make sure to finish above steps within this period. If a longer working time is desired, store the powder and liquid in a refrigerator for more than 4 hours.

- Remove the impression from the mouth before the exothermic reaction starts, approximately 1 minute and 30 seconds from start of mixing. At this stage, material still has a rubbery consistency.
- Follow the UNIFAST III to be 30 sure in the impression. The setting time is 3 minutes 10 seconds from start of mixing.

- Remove the temporary appliance from the impression.
- Adjust and finish in a usual manner. Carefully fit in mouth and establish proper marginal adaptation and occlusion. The margins can be relied by adding a layer of freshly mixed UNIFAST III.

NOTE:
In case of making a temporary bridge using pre-fabricated temporary crowns as pontics, immerse the bridge in water at 50°C for 5 minutes or leave undisturbed at room temperature for 30 minutes before placing it in the patients' mouth to ensure complete setting.

- After final finishing, lute the inlay/crown/bridge with a temporary cement like FREEGEnOL TEMPORARY PAKC.

CLINICAL HINT

The surface of the temporary appliance can be coated with G-COAT PLUS for a high gloss esthetic result. Refer to the respective manufacturer's instructions for use.

2. Direct Method – multi shade standard technique
- 1) Use a tooth matching Vita shade in the technique as described above (Direct method, single shade technique 1-8).
- 2) Trim the incisal area of the temporary restoration.
- 3) Apply an incisal, enamel trimmer on the incisal area, reusing the impression or by applying with a brush (brush-on technique).
- 4) Continue with Direct method 9.
- 5) Indirect Method – single shade standard technique
- 1) Fabricate a stone model using a standard technique and use an impression / silicone index.
- 2) Wax-up the restoration using a standard technique and use an impression / silicone index.
- 3) Coat the stone model with a separator such as ACROSEEP or MULTI-SEP.
- 4) Dispense the desired amount of UNIFAST III powder in a rubber cup and add liquid, quickly mix with a plastic spatula for 10-15 seconds. Immediately pour the mixture into the desired area of the impression/silicone index. Standard powder to liquid ratio is 1 g of powder to 0.5 mL of liquid (1 scale of powder to 1 scale of liquid). Powder to liquid ratio is adjustable between 2 g : 1 mL and 2 g : 1.5 mL.
- 5) Wait until the mixture reaches a soft dough stage.
- 6) Place the impression on the model with pressure.

NOTE:
UNIFAST III starts to set 1 minute after mixing. Make sure to finish above steps within this period. If a longer working time is desired, store the powder and liquid in a refrigerator for more than 4 hours.

- 7) Allow the material to set. Setting time is 3 minutes 10 seconds from start of mixing.
- 8) Remove the temporary appliance from the impression/silicone index.
- 9) Clean the inner surface of the temporary appliance using a cotton pellet soaked with ethanol.

- 10) Adjust and finish in a usual manner. Carefully fit in the model and establish proper marginal adaptation and occlusion. The margins can be relied by adding a layer of freshly mixed UNIFAST III.

NOTE:
In case of making a temporary bridge using pre-fabricated temporary crowns as pontics, immerse the bridge in water at 50°C for 5 minutes or leave undisturbed at room temperature for 30 minutes before placing it in the patients' mouth to ensure complete setting.

CLINICAL HINT

The surface of the temporary appliance can be coated with OPTIGLAZE for a high gloss esthetic result. Refer to the respective manufacturer's instructions for use.

- 1) After final finishing, lute the inlay/crown/bridge with a temporary cement like FREEGEnOL TEMPORARY PAKC.
4. Indirect Method – multi shade standard technique
- 1) Use a tooth matching Vita shade in the technique as described above (Indirect method, 1-8).
- 2) Trim the incisal area of the temporary restoration.
- 3) Apply an incisal, enamel trimmer on the incisal area, reusing the impression/silicone index or by applying with a brush (brush-on technique).
- 4) Continue with Indirect method 9.

5. Repairing temporary appliances

In case a repair is needed on a temporary appliance, remove the temporary appliance from the mouth and immerse the fractured surfaces and adjacent areas. Undercuts should be provided. Apply mixed material and allow the material to set. For details see above.

NOTE:
Repairs should not be done in the mouth.

B. REPAIR OF A FRACTURED DENTURE

- 1) Take an impression of the edentulous area in a usual manner and fabricate a stone model.
- 2) Roughen the fractured surfaces and adjacent areas of the denture using a bur. When the denture cannot be repaired, the metal surface should be roughened with a carbide bur, sandblasted, cleaned with air and treated with METALPRIMER II.
- 3) Coat the stone model with a separator such as ACROSEEP or MULTI-SEP and place the fractured denture base on the stone model.
- 4) Dispense the desired amount of UNIFAST III powder in a rubber cup and add liquid, quickly mix with a plastic spatula for 10-15 seconds. Immediately pour the mixture into the fractured area of the denture base. Standard powder to liquid ratio is 1 g of powder to 0.5 mL of liquid (1 scale of powder to 1 scale of liquid). Powder to liquid ratio is adjustable between 2 g : 1 mL and 2 g : 1.5 mL.
- 5) Wait until the mixture reaches a soft dough stage.
- 6) Place the impression on the model with pressure.

NOTE:
UNIFAST III starts to set 1 minute after mixing. Make sure to finish above steps within this period. If a longer working time is desired, store the powder and liquid in a refrigerator for more than 4 hours.

- 5) Setting time is 3 minutes 10 seconds from start of mixing.
- 6) After set, adjust, trim excess material and polish in the usual manner.

CLINICAL HINT – Brush on technique

As an alternative to mixing powder and liquid in a rubber cup, dispense the desired amount of powder and liquid on the surface of the fractured area of the denture with the liquid and pick up a small amount of powder. Deposit this mixture on the surface and progressively build up the temporary appliance, pattern or repair by repeating the procedure. With this technique, the setting time is 3 minutes 30 seconds from start of mixing. Clean the brush with liquid after use or during the work if necessary.

IMPORTANT NOTE

An exothermic reaction will occur during setting. When the uncured material is placed in the mouth, remove before the temperature starts to increase to avoid burn or necrosis of pulp.
Do not use UNIFAST III to repair denture materials or to replace pre-fabricated temporary crowns made out of polycarbonate, polysulfone, or polyethersulfone resin.
The amount of restorative material can be increased by the procedure. The temporary appliance in water for at least 10 minutes prior to luting in the patient's mouth. A small or thin portions of mixed material may not set properly. Make sure mixed materials are fully cured before placing/luting in the mouth.

SHADES

AI, A2, A3, A3.5, B2, E3, Incisal, Clear, No.3 (Pink), No.8 (Pink Veined)

PACKAGES

UNIFAST III 2-1 Package
a. Powder 35 g (in all shades)
b. Powder 250 g (in all shades)
c. Powder 250 g (in all shades)
d. Powder 300 g (in all shades)
e. Liquid 40 g (42 mL)
f. Liquid 100 g (104 mL)
g. Liquid 250 g (260 mL)
h. Plastic Case
i. Disposable dispensing dish No.2 (60)
j. Brush No.10 (1)
k. Plastic Case
*Only available in limited countries.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

CAUTION

1. The liquid is flammable. Avoid sources of ignition. Do not keep or use near fire or high temperature.
2. Avoid direct contact of powder, liquid or mixture on skin. In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.
3. Avoid direct contact of powder, liquid or mixture on eyes. In case of contact, immediately flush with water.
4. The liquid is volatile. Use in a well ventilated area. If mixed material is placed in the mouth before set, advise patients to breath through the nose.
5. Do not pour the powder or liquid remaining in the rubber cup back into the bottle.
6. Do not mix UNIFAST III with UNIFAST Trad or any other acrylic resin products.
7. Do not use a brush or rubber cup which are used with other acrylic resin products. This may delay or inhibit setting.
8. Wear a dust-protective mask when finishing and polishing the set material.
9. Liquid may turn yellow with time, but this will not affect its function.
10. The liquid should not be left in the rubber cup for prolonged period of time as it changes the handling property.
11. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
12. In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gdental.europa>

They can also be obtained from your supplier.

CLEANING AND DESINFECTING
MULTI-USE DELIVERY SYSTEMS are designed to avoid cross-contamination between patients this requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERS. Thoroughly clean device to prevent drying and accumulation of contaminants.

Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory@gd-eu.com

Undesired effects: Reporting
If you become aware of any undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following links:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system gd@gdental.eu. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM LTD
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 11/2025

EU: GC EUROPE N.V.
T: +31 (0)20 678 8200
F: +31 (0)20 678 8201
E: info@gd-eu.com

GC ASIA DENTAL PTE. LTD.
1 Temasek Centre 1, #01-01 Temasek Place 2, Singapore 039541
T: +65 6946 7568
F: +65 6946 7569
E: info@gd-asia.com

GC AUSTRALIA DENTAL PTY. LTD.
1752 Botany Rd, Bankstown NSW 2190, Australia
T: +61 (0)2 9501 8200
F: +61 (0)2 9501 8201
E: info@gd-australia.com

PRINTED IN JAPAN

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
1-880 Tamauchi-cho, Maebashi, Saitama 350-0844, Japan

DISTRIBUTED BY
GC CORPORATION
1-1 Naumachi-cho, Tsubati-Nu, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
T: +31 (0)20 678 8200
F: +31 (0)20 678 8201
E: info@gd-eu.com

GC ASIA DENTAL PTE. LTD.
1 Temasek Centre 1, #01-01 Temasek Place 2, Singapore 039541
T: +65 6946 7568
F: +65 6946 7569
E: info@gd-asia.com

GC AUSTRALIA DENTAL PTY. LTD.
1752 Botany Rd, Bankstown NSW 2190, Australia
T: +61 (0)2 9501 8200
F: +61 (0)2 9501 8201
E: info@gd-australia.com

PRINTED IN JAPAN

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
1-880 Tamauchi-cho, Maebashi, Saitama 350-0844, Japan

DISTRIBUTED BY
GC CORPORATION
1-1 Naumachi-cho, Tsubati-Nu, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
T: +31 (0)20 678 8200
F: +31 (0)20 678 8201
E: info@gd-eu.com

GC ASIA DENTAL PTE. LTD.
1 Temasek Centre 1, #01-01 Temasek Place 2, Singapore 039541
T: +65 6946 7568
F: +65 6946 7569
E: info@gd-asia.com

GC AUSTRALIA DENTAL PTY. LTD.
1752 Botany Rd, Bankstown NSW 2190, Australia
T: +61 (0)2 9501 8200
F: +61 (0)2 9501 8201
E: info@gd-australia.com

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