Prior to use, carefully read the instructions for use.

EN

GC TISSUE CONDITIONER

TISSUE CONDITIONING AND RELINING MATERIAL.

DESCRIPTION
GC TISSUE CONDITIONER is a tissue conditioning and relining material for existing removable complete and partial dentures. Thanks to its patented technology, the material remains soft over time.
This product is a Type B (Extra soft), Class 1 (chemical free) material according to classification of ISO 10139-1. Medium nature of system is plasticized acrylics.

INTENDED PURPOSE

- Temporary relining
- Tissue conditioning
- Functional impression taking

INDICATIONS FOR USE

- Improper fit of acrylic dentures and discomfort.
- Areas of application:
 - Misfitting dentures needing temporary relining
 - Wounded soft tissues
 - Impression for making new denture or short-term relining of existing ones

CLINICAL BENEFITS

Improvement of denture fit and comfort while supporting tissue healing.

CONTRAINDICATIONS

Patients with known allergies to methacrylate monomer, methacrylate polymer, ethanol or aliphatic acid ester-based plasticizer.

INTENDED USERS/SPECIAL TRAINING

- Dentists
- Denturists
- No special training required

PATIENT TARGET GROUP

Target populations are edentulous or partially edentulous patients in need of relining for acrylic dentures or tissue conditioning.

PERFORMANCE CHARACTERISTICS

The product's performance characteristics satisfy the requirements of the intended use.

DIRECTION FOR USE

- PREPARATION**
Check the patient's oral conditions and the denture in use. Adjust misfitting areas with a fluoride bur, clean and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain the elasticity.
Apply COCOA BUTTER or petroleum jelly to the areas where the denture should not be relined.
 - POWDER / LIQUID DISPENSING**
The standard powder / liquid ratio is 2.4 g / 2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup.
Note:
 - Using less liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.
 - Powder may form aggregates in the bottle under certain storage conditions. This will not affect the function.
 - Close the bottles of powder, liquid and coating agent tightly immediately after use.
 - MIXING**
Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.
 - APPLICATION**
1) The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only on the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand.
2) Apply the mixture evenly to the lubricating surfaces of the prepared denture.
Note:
 - To avoid sticking of the material, lubricate fingertips or instruments with water. Do not use lotion or petroleum jelly.
 - Prevent excess from flowing into the throat.
 - Seal the denture in the mouth. Ask the patient to gently occlude in the centric occlusion and move the lips and mandibles to move front. After keeping the denture for about 5 minutes in this position, remove trim and inspect for excess and deficiencies.
 - Remove any excess using a sharp instrument.
 - FINISHING**
1) Wash away saliva from the relined denture under running water. Dry with an air syringe.
2) Apply the coating agent to the relined surfaces of the denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 4-5 minutes.
Note:
 - To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER / denture interface.
 - Reseat the denture in the mouth.
 - Note:
 - Immersion of the denture in water before seating in the mouth will reduce unwanted odor.
 - Instruct the patient to clean the relined areas with fingers under running water. The use of a brush may damage the relined areas. Care should be taken not to use denture cleaning agents for a prolonged period of time.
 - REMOVAL**
It is recommended to remove the material or replace it with a new one after a few days or weeks depending on each clinical case.
GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.
- DISPOSAL**
Dispose of all waste according to local regulations.
- STORAGE**
Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).
- CLEANING AND DISINFECTING**
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERGE: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.
- SIDE EFFECTS**
With the use of this material, sensitivity such as rubor, erosive inflammation, water blister, edema may occur.
- RESIDUAL RISKS**
There are no known residual risks to date, other than those specified as part of contraindications, limitations, precautions, warnings, and side effects.
- PRECAUTION, WARNINGS AND LIMITATION OF USE**
Do not use if the prescribed application technique cannot be applied.
1. The liquid is flammable. Avoid sources of ignition. Do not keep or use near flames or high temperature.
2. Use in a well ventilated place. Close bottles immediately after use.
3. Avoid contact with eyes. In case of contact, immediately flush with water and seek medical attention.
4. Avoid direct contact of liquid, mixture or coating agent on skin.
5. In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.
6. Coating agent may turn yellow with time, but this will not affect its function.
7. Do not mix with other products.
8. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
9. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<https://www.gc.dental/europe>
They can also be obtained from your supplier.
- PACKAGE**
1. 1-1 Package
Kit contains: Powder 90 g (1), Liquid 90 g (101 mL (1)), Coating agent 12 g (15 mL (1)), Powder measure (1), Plastic liquid syringe No.1 (1), Brush No.7 (1), Plastic spatula (1), Rubber cup (Large) (1)
2. Refill Packages
a. GC TISSUE CONDITIONER powder 90 g (in each shade)
b. GC TISSUE CONDITIONER liquid 90 g (101 mL)
c. GC TISSUE CONDITIONER coating agent 12 g (15 mL)
- SHADE**
Live pink, White
- REPORTING OBLIGATION ACCORDING TO EU MDR**
In the case of serious incidents related to the device, please report it to GC Europe N.V. (vigilance@gc.dental) and to the competent authority of the Member State in which the user/patient resides.
- Note:**
The current instructions for use are available in the download section of the <https://www.gc.dental/ website>. To access the appropriate version, please select your region from the footer.
- UK Responsible Person
GC UNITED KINGDOM Ltd.
Coppers Court Newport Pagnell, Buckinghamshire, MK16 8US, United Kingdom

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