

Boost Your Dental Practice's Hygiene with

CliniMax™

– Ideal for Instruments & Equipment!



Cost effective

Excellent in hard water

Low foaming formula

Bacteriostatic, prevents the growth of bacteria

Gentle on high quality instruments

For manual instrument cleaning

Whiteley 

CliniMax™

INSTRUMENT AND EQUIPMENT DETERGENT

For Medical, Dental and Laboratory Use

CLINIMAX™ uses the latest surfactant technology to emulsify and solubilise organic soils such as blood, protein and lipids (fats).

CLINIMAX™ is a mildly alkaline, non-hazardous, multipurpose detergent for Medical and Dental applications that is specifically formulated to meet the requirements of AS/NZS 4187:2014 and AS/NZS 4815:2006.

CLINIMAX™ can be used for Manual and Ultrasonic Instrument Cleaning, Equipment Cleaning and also for the cleaning of environmental surfaces such as surgical tables, lighting, benches and trolleys.

USE OF GLOVES IS RECOMMENDED WHENEVER USING DISINFECTANTS AND DETERGENTS

5 Litres

Whiteley 

CliniMax™

INSTRUMENT AND EQUIPMENT DETERGENT

For Medical, Dental and Laboratory Use

CLINIMAX™ uses the latest surfactant technology to emulsify and solubilise organic soils such as blood, protein and lipids (fats).

CLINIMAX™ is a mildly alkaline, non-hazardous, multipurpose detergent for Medical and Dental applications that is specifically formulated to meet the requirements of AS/NZS 4187:2014 and AS/NZS 4815:2006.

CLINIMAX™ can be used for Manual and Ultrasonic Instrument Cleaning, Equipment Cleaning and also for the cleaning of environmental surfaces such as surgical tables, lighting, benches and trolleys.

USE OF GLOVES IS RECOMMENDED WHENEVER USING DISINFECTANTS AND DETERGENTS

Dr. Greg S. Whiteley

ARTG No: 125528

Directions for use:

Do not mix with other detergents or chemicals.

Manual Cleaning:

1. Use in warm water.
2. Remove gross visible soil under running water.
3. Use 4-6mL of CliniMax™ per litre of water – 20-30mL per 5L.
4. Clean instruments, equipment or laboratory apparatus and remove organic soil.
5. Dispose of solution when dirty (cloudy) or within 24 hours of dilution.

Hard Surface Cleaning:

1. Use in warm water.
2. Ensure mildly alkaline detergents are suitable for the surface to be cleaned.
3. Use 4-6mL of CliniMax™ per litre of water and clean.
4. Dispose of solution when dirty (cloudy in bucket) or within 24 hours of dilution.

Packsize: 500mL empty bottle
12 x 1L
2 x 5L

Product Code: 030406
030402
030404

*Pump for 1L bottle sold separately.



Find Out More



Australian Made
Fast local supply