MICRODONT

DENTAL PRODUCTS

RMP / DATA SHEET

POP 04 F 09 – Data de Implantação: 01/07/2024 – Rev. 02

Fabricante / Manufacture / Fabricante

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Trade Name: Zirconia Rotating Tip – Soft Bur

Technical Name: Dental Surgical Drills

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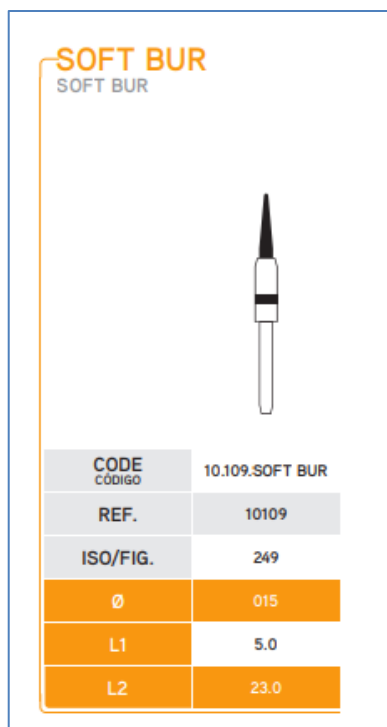
Product indicated for interventions in: gingival contour, removal of the gingival sulcus for impression taking and periodontal treatment, exposure of intraosseous implants and impacted teeth, exposure of deep cervical caries and elimination of granulation tissue, gingival hyperplasia.

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The ceramic tip is manufactured for performing soft tissue surgery, its advantage is safe cutting without heating, which promotes blood clotting, reducing the tendency to hemorrhage as well as preventing tissue necrosis.

The Zirconia Rotating Tip is a high-speed instrument made of Zirconia porcelain and AISI 303 Stainless Steel, it is used by professionals in the dental area to perform soft tissues and safe incisions, being individually packaged in an aluminum Broquet, in unopened condition. sterile and properly identified.

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AISI 303 stainless steel

Ceramic tip - sintered in zirconium oxide

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Microdont Soft Bur is recommended for: soft tissue surgery, such as: exposure of implants or gingivectomies. The benefits of our ceramic bur are: reduced hemorrhage chances and avoiding tissues necrosis.

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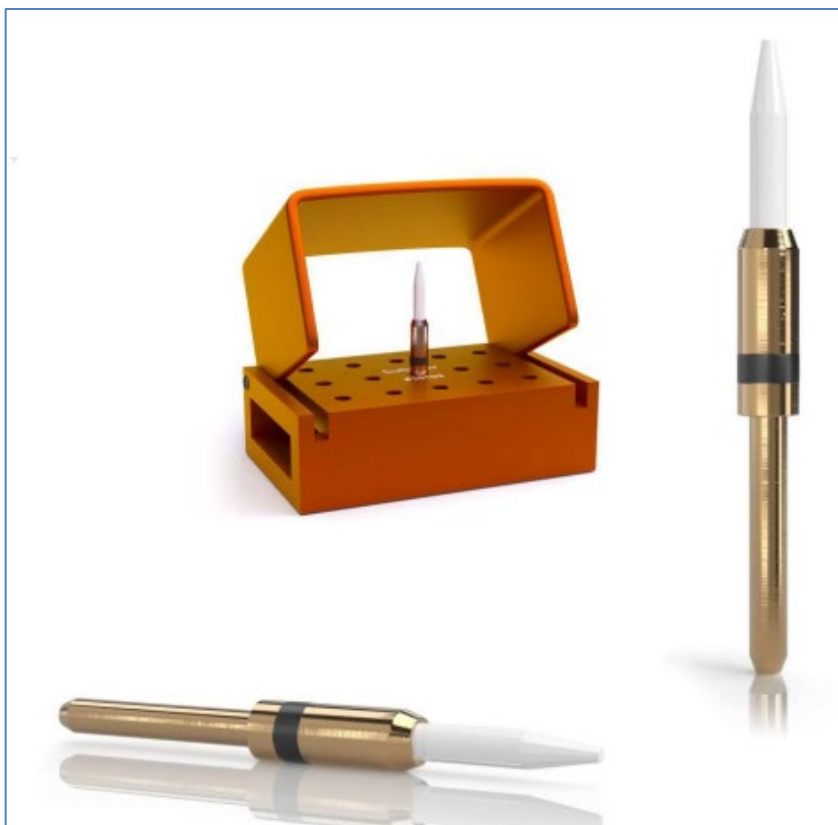
THE CERAMIC TIP is an instrument as fragile as other surgical instruments, requiring special care during its sterilization and handling, as well as in its conservation, which must be protective against falls and shocks.

The product must be in an airy, dry place and away from light in order to maintain ideal conditions for storage and transport, as well as its physical and chemical integrity, handling should only be done by professionals in the dental field.

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-During use the turbine should be driven at high rotation of 300,000 to 500,000 rpm, lower speeds can cause injury.

- The use of the refrigerator spray during work can increase the tendency to hemorrhage by avoiding coagulation of capillary vessels.

- Before use check that the tip is damaged, avoiding possible damage.

- Improper cutting or inefficiency may occur if the active part of the tip is damaged.

- Before using make sure the drill is properly seated in the handpiece.

- The professional must use all personal protective equipment.

- This product is for the exclusive use of dentists, as they have the necessary scientific information for its proper use.
- Manufacturer recommends single use.
- Sterilizable in autoclave. Autoclave re-sterilization: 121° for 20 minutes.

Attention:

Perigo microbiological contamination if re-sterilization is not done correctly. The reuse of the device wears out the abrasive surface, causing a decline in its performance, only 5 reprocessing is recommended.

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3R uses the following documents in its QMS to guarantee the manufacturing quality of its products.

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- IT 59 – Soft Bur;
- POP 04 – Project control;
- POP 08 – Identification and traceability;
- POP 14 – Inspection and Testing;
- POP 17 – Handling and Storage;

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ISO14971: medical devices - application of risk management to medical devices

ISO 13485: medical devices-quality management systems-requirements for regulatory purposes.

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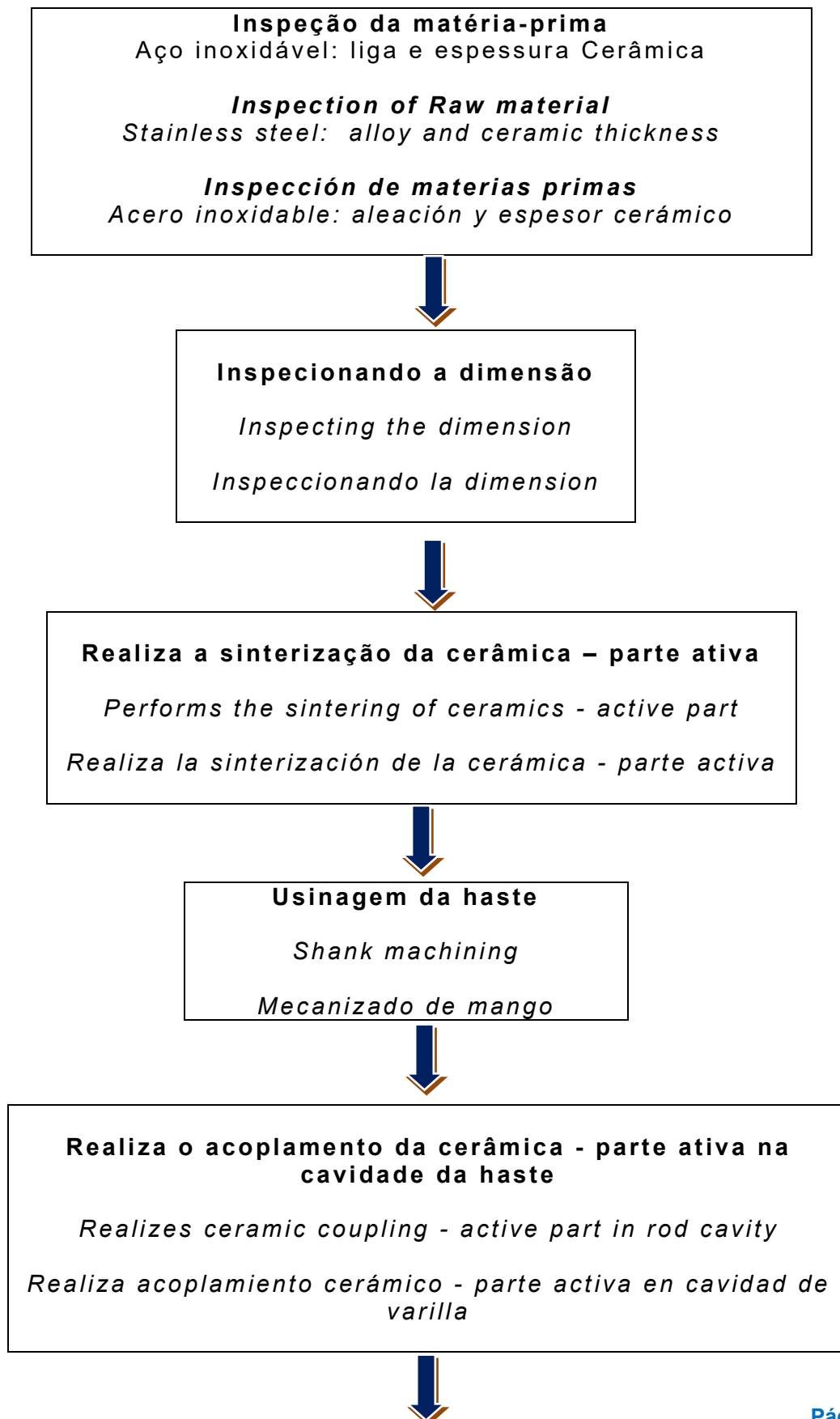
Non-perishable product. Not applicable.

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Dressing - sharpening and formatting of ceramics

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Quality control

- *Eccentricity control of ceramics: beat - Clock feeler;*
- *Rod machining control: dimensional - micrometer and caliper;*
- *Eccentricity control of the ceramic tip - fixing the glue;*
- *Approval of the visual aspect - electron microscope.*

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Cleaning

wash with neutral detergent and rinse under running water

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Final inspection
Visual Method:
*to check for imperfections in the
diamond surface (active area)*
Sampling 100% of products

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Important warning:

1. All production processes are carried out by 3R Industria e Comércio Ltda.
2. Design and development are carried out by 3R Industria e Comércio Ltda.

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According to the attached file I-certified quality management system: EN ISO 13485
certificate

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