ENGLISH

Indication:

The Soft Bur is indicated to perform oral soft tissue surgery and assist in periodontal treatments, such as: gingival contour (gingivectomy), exposure of intraosseous implants, elimination of granulation tissue, gingival hyperplasia and papillectomy, exposure of included and semi-included teeth, exposure of deep cervical caries and removal of the gingival sulcus for printing.

The company is not responsible for the use of the product outside the intention of proposed use.

Fundamentals of its operation and action:

Ceramic allows the vessels to clot and reduces the tendency to hemorrhage.

It provides high stability, allowing it to act on the gums quickly and reliably. In addition, minimal heating prevents necrosis during treatment.

Instructions for Use:

- Dry the region well, the surface of the fabric should be dry.
- Check that the refrigeration is turned on, in this case we do not use refrigeration. It is important not to cool with water spray during use, the use of refrigerator spray can increase the tendency to hemorrhage.
- Snap the tip to the high-speed pen
- With the tip actuated apply to the mucosa to be removed in a rotation between 300,000 to 500,000 rpm with maximum speed. A lower turning speed can cause injury and should therefore be avoided.
- Insert the tip into the fabric and make the cut. At this speed the kinetic energy is converted into heat when in contact with the tissue, the tip does not heat up because it is made of thermo-resistant material, the heat is transferred to the tissue that immediately restings the bleeding (hemostasis).

Manual cleaning procedure:

- Use a reserved area specifically to perform cleaning.
- Rinse the tip in running water and keep it immersed in a bath of enzymatic detergent solution, following the manufacturer's instructions.
- After soaking and keeping it immersed, brush well away from the body with the neutral cleaning agent. Care should be taken to avoid spreading contaminants from the spray during brushing procedures. Do not use wire brushes.
- Special care should be taken to clean the cracks and other hard-to-reach areas. Inspect visually to confirm debris removal. Repeat the cycle if necessary.
- Rinse the tip thoroughly under running water until it is visibly clean.
- Dry the device with clean compressed air or paper towel.

Ultrasonic cleaning procedure

- Prepare an enzymatic detergent solution by following the agent manufacturer's instructions for the correct concentration, exposure time, temperature and water quality.
- Place the drill in the proper place of the ultrasonic equipment, following the instructions for use of the equipment manufacturer
- Completely immerse the device in the cleaning solution and program the equipment for at least fifteen (15) minutes.
- Rinse the drill well under running water until it is visibly clean.
- Dry the device with clean compressed air or paper towel.

Sterilization Requirements:

It is recommended to sterilize the Carbide Bur by pressure saturated vapor autoclave at 121°C for 20 minutes, properly packed in its own autoclave packaging, or sterilization parameters should be followed according to relevant policies/rules of the country where the device is being used.

It is the professional's total responsibility to keep the autoclave equipment up to date to ensure the correct sterilization of the product.

Warnings and Precautions:

- Before use, check that the tip is in perfect conditions of use.
- Non-sterile product.
- Sterilize the product before use.
- It is important not to use pen refrigeration, as refrigeration can increase the tendency to hemorrhage, preventing blood clotting.
- Rotations below 300,000 rpm can cause injury
- During operation, the professional must use all personal protective equipment, including protective eyewear.
- Wear protective eyewear on the patient.
- The Soft Bur is considered a contaminated instrument and proper handling precautions are required. Use PPE, including gloves, masks and safety glasses when handling, as with all other dental instruments.
- LOT number: It is important to keep the lot number, identified on the product label, to be informed in eventual occurrences with the manufacturer.
- Disposal of material must follow relevant local legislation.
- Keep the package in a dry place, protected from light and away from bad weather. Keep products in their original packaging until use.
- Before use, check that the product is in perfect conditions of use, especially in relation to the integrity of the rod.

Conservation and Storage:

- Keep the product in its original packaging until use. Inspect the packaging before use.

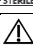
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
• The product must be stored in an airy, dry place, protected from light, in order to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.

Other Additional Information:

- If necessary, 3R provides the instructions for use in printed format, at no additional cost, upon request by e-mail (administrativo@3r.ind.br) or by phone: (+55 11-5525-0590 / +55 0800-773-5525), and we will return within a maximum of 7 working days from the request.
- Always check on the site: www.microdont.com.br, the most up-to-date version of the usage instruction corresponding to the product.

Composition: Ceramic and stainless steel bur

Symbology:

Symbol	Symbol Title	Description
	Expiration / Use-by date	Indicates the date after which the health product should not be used.
	Consult instructions for use	Indicates the user's need to refer to the instructions for use.
	Date of manufacture	Indicates the date when the health product was manufactured
	Manufacturer	Indicates the Manufacturer of the product for the health as defined in the Directive of the European Community 90/385EEC, 93/42/EEC and 98/79/EC.
	Batch Code	Indicates the manufacturer's batch code so that the batch or departure can be identified.
	Catalog number	Indicates the manufacturer's reference number so that the product for the health can be identified.
	Sterilized using irradiation	Indicates that the product is sterilized
	Non-Sterile	Indicates that the product is not sterile
	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself

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This review cancels and replaces the previous revision.

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