



1. From the Main Screen, click the down arrow in the top right corner.




5. Enter in the desired file name and select the check button

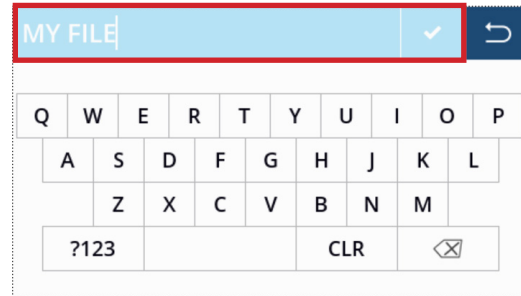


2. Then select the pencil icon next to custom sequence.

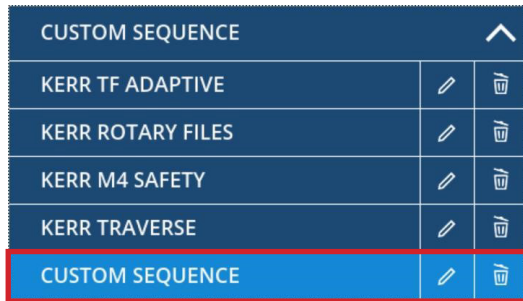


6. Select the "-" Section of the row and select the pencil icon next to it

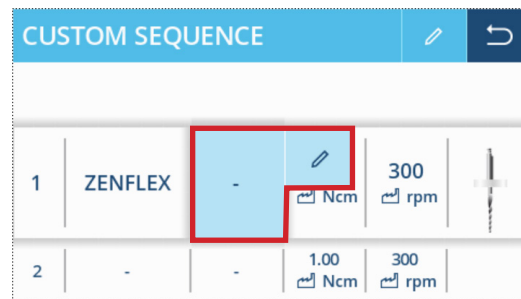


3. Select the "My File" section of the row

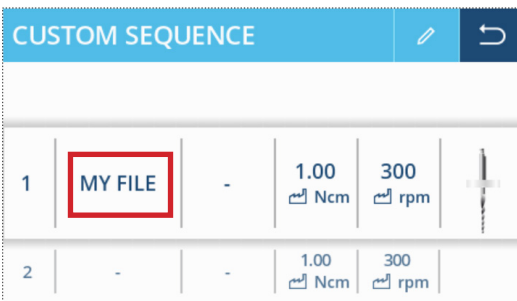


7. Enter in the size of the file and select the check button

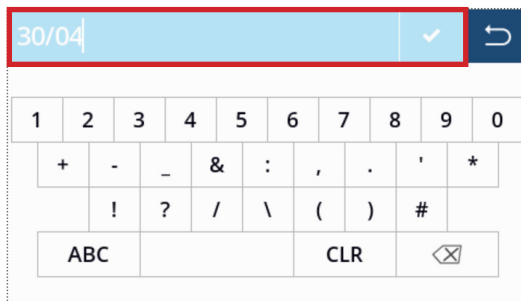

4. Select the pencil icon next to the "My File" section

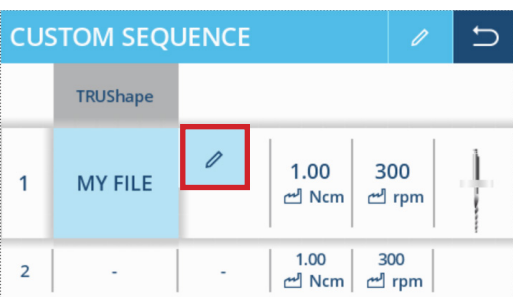


8. Select the Torque Value Section of the row and scroll to set desired Torque Value (value recommended in IFU per file size)

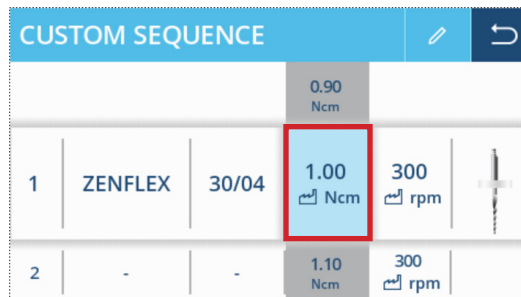


9. Select the RPM Section of the row and scroll to set desired RPM Value (value recommended in ZenFlex IFU per file size)



13. Enter in the desired sequence name and select the Check button

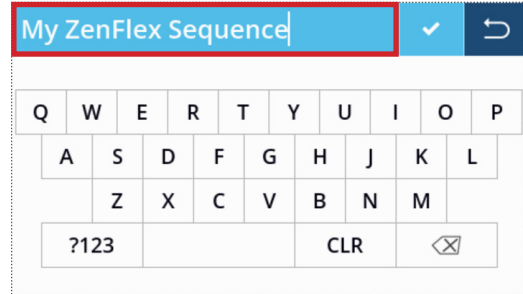

10. Select the File section of the row and the scroll to set the corresponding Tip Size band color

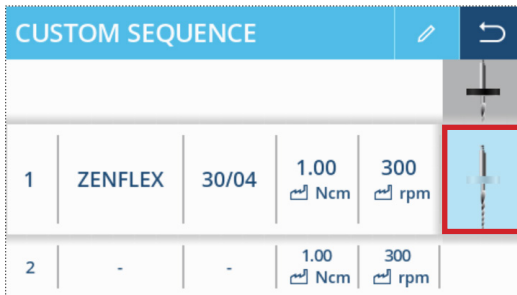


14. Select the Return Arrow in the top right corner to save the sequence and return to the main screen



11. Repeat steps #3-10 as needed to generate the full desired sequence

12. Select the Pencil Icon in the top right corner

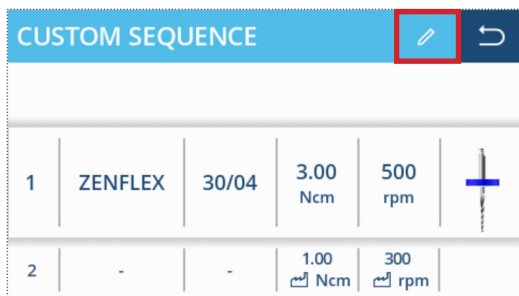

15. The preset is now saved under the desired name and settings



To learn more about ZenFlex™ visit our website at: [go.kerrdental.com/zenflex](http://go.kerrdental.com/zenflex)