

# Directions For Use

**Caution:** For Professional Use Only.

## Ultima Needle Re-Capper



### Indications For Use:

The Ultima Needle Recapper is a receptacle for holding the needle cap for one-handed recapping after injection.

Use of the Ultima Needle Recapper can prevent accidental needle stick injuries and eliminate a source of cross contamination of infectious diseases.

### Precautions:

Recapping step must be carried out one-handed to prevent the possibility of needle sticks.

### Step-By-Step Instructions:

1. Remove needle cap and place into the Ultima Needle Recapper. Press down the cap and it will be securely held.

2. For recapping - Place Ultima Needle Recapper on a stable, solid surface. With the needle cap resting in the receptacle (from step 1), use one hand to guide the syringe with the uncapped needle down into the cap.

### DO NOT HOLD THE Ultima device during re-capping

3. If re-injection is required, repeat steps 1 and 2.

4. Use one hand to guide the syringe with the uncapped needle down into the cap and then press the button as shown to release the needle cap.

### Dispose of needle in compliance with local laws



**Step 1**



**Step 2**

